READ THIS MANUAL COMPLETELY AND CAREFULLY BEFORE USING THIS PRODUCT

Keep this manual in a safe location for future reference

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CONTENT

1. IMPORTANT SAFETY INSTRUCTIONS.....	2	6. Care and Maintenance	9
2. Introduction	4	7. Troubleshooting Guide.....	9
3. Parts and Controls.....	5	8. Specifications	10
4. To Set Up	5	9. FCC Compliance Declaration	11
5. To Use	6	10. Warranty	11

SAVE THESE INSTRUCTIONS FOR FUTURE REFERENCE

If the instruction manual is damaged or if you no longer have the instruction manual in your possession, please contact Beurer customer service. Please see warranty for service contact.

Dear Customer,

Thank you for choosing this Beurer product. Our name stands for high-quality, thoroughly tested products for applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage and air.

Please read these instructions completely and carefully before using this product and keep them for later reference. Be sure to make them accessible to other users and observe the information they contain.

With best regards,
Your Beurer Team

Indication for use

The ear thermometer is intended for measuring the body temperature in the ear. Do not use the ear thermometer for measurements in other parts of the body.

You can also use the ear thermometer to measure room and object temperatures.

1. IMPORTANT SAFETY INSTRUCTIONS

Signs and symbols

The following signs appear in the Safety Section (page 2) and in this manual on page 2.

READ AND UNDERSTAND THIS ENTIRE MANUAL INCLUDING THE SAFETY SECTION AND ALL INSTRUCTIONS AND WARNINGS COMPLETELY AND CAREFULLY BEFORE USING THIS PRODUCT. FOLLOW ALL SAFETY INSTRUCTIONS AND WARNINGS TO AVOID HAZARDOUS SITUATIONS AND TO MAKE CORRECT USE OF THIS PRODUCT.

Whenever used, the following identify safety and property damage messages and designate a level of hazard seriousness.

	WARNING Warning notice indicating a risk of injury or damage to health.
	CAUTION Safety note indicating possible damage to the unit/accessory.
	NOTICE addresses practices not related to personal injury, such as product and/or property damage.
	DANGER indicates a hazardous situation, which, if not avoided, will result in death or serious injury. Usage of this signal word is limited to the most extreme situations.

	Note Note on important information.
	Observe the instructions for use
	Application part, type BF
 Storage -13 °C 131 °F RH 15-95%	Permissible storage temperature and humidity
 Operating 50 °F 10 °C 104 °F 40 °C RH 12-85%	Permissible operating temperature and humidity
	Serial number
IP 22	IP22, protection against harmful ingress of water and dust

CAUTION – To reduce the risk of personal injury or product/property damage:

- This device is designed solely for non-commercial, household use in non-invasively measuring temperatures via an infrared sensor of the auditory canals of adults and children, as well as detecting surface temperatures of objects and ambient temperatures of surrounding areas. Do not use for any other purpose.
- Do not disassemble this unit.
- This thermometer is a sensitive electronic device. Treat it carefully and do not expose it to any sharp impacts.
- Do not expose the thermometer to direct sunlight.
- Place thermometer in the room where measurements will be made for at least 30 minutes before use.
- The thermometer is NOT waterproof; keep away from water and other liquids.
- Clean the measuring tip after each use with a soft cloth moistened with disinfectant.
- Before each use, ensure the sensor is undamaged; otherwise, contact customer service.
- This device is not intended for commercial or clinical use.
- Different measurement results are possible between the left and right ears. In order to record temperature changes, always measure in the same ear for the same person.
- If you have been lying on one ear for some time, if your ears have been covered or exposed to heat or cold, you have just been swimming or bathing, measurements will be inaccurate; wait about 20 minutes before measuring.
- Replace the protective cap after each use to ensure an accurate measurement and prevent crosscontamination.
- The ear thermometer may only be used with protective caps from Beurer; other protective caps may result in faulty measurements. If you have run out of protective caps, contact the manufacturer or retailer for replacements.
- Ear wax can affect the measurement; ensure ear is clean before measuring.
- Do not use on children under six months of age. In infants under six months, the ear passage is still very narrow and the temperature of the eardrum can often not be recorded, so any results obtained may be inaccurate.
- The measurement must not be taken in an ear affected by inflammatory diseases (e.g. discharging pus or secretion), after possible ear injuries (e.g. eardrum damage) or after any ear operation while the ear is healing.
- The body temperature measured in the ear may differ from other measurements taken in the mouth, rectum or armpits. Any comparison of these values is therefore futile.

- Measure temperature regularly to calculate the normal ear temperature and then use these measurements as a basis for comparison when taking measurements in the event of a suspected high temperature.
- Do not use the thermometer on multiple people if any of them have an ear infection, even when using protective covers on the sensor tip.
- Use this ear thermometer only with the protective cover in place.
- Read these instructions for use carefully and keep them for later use.

NOTE:

- Taking your temperature provides information about your current state of well-being. If you are uncertain about interpreting the results or if the result is abnormal (e.g. fever), consult your doctor. This also applies in the case of slight temperature changes if there are other symptoms of illness such as agitation, severe sweating, flushed skin, fast pulse rate, tendency to collapse, etc.
- Temperatures measured with different thermometers cannot be compared. You should tell your doctor or keep in mind if you are measuring for yourself what type of thermometer you use and in which part of the body.

Battery Handling Safety Precautions

- Use only the size and type of batteries specified.
- Be sure to follow the correct polarity when installing the batteries. Reversed batteries may cause damage to the device.
- Do not mix different types of batteries together (e.g., alkaline and carbon-zinc or rechargeable batteries) or old batteries with fresh ones. Always replace batteries as a simultaneous set.
- If the batteries in the device are depleted or the device will not be used for a long period of time, remove the batteries to prevent damage or injury from possible battery leakage.
- Do not try to recharge batteries not intended to be recharged; they can overheat and rupture (follow battery manufacturer's directions.)
- Do not dispose of batteries in fire, batteries may explode or leak.
- Clean the battery contacts and also those of the device prior to battery installation.
- Remove discharged batteries from the product and dispose/recycle in compliance with all applicable laws.
- Swallowing batteries and/or battery fluid can be extremely dangerous. Keep the battery out of the reach of children. Should a child, or any other person swallow a battery and/or battery fluid, please call 911 immediately.
- Should battery fluid leak and come into contact with your eyes or skin, immediately rinse with plenty of clean water. Please call 911 immediately.
- Batteries should not be charged or reactivated by any other means.
- Batteries should not be taken apart, thrown in the fire or short circuited.
- Keep batteries away from children and pets. Batteries may be harmful if swallowed. Should a child or pet swallow a battery, seek medical assistance immediately.

2. Introduction

This infrared thermometer measures:

- Temperature in the human ear
- Surface temperature of objects and liquids
- Ambient temperatures


Additional features include:

- Stores last 10 measurements for plotting changes in temperature.
- Stores date and time with each measurement.
- Fever alarm for temperatures over 100,4 °F (38 °C).
- Switchable between °F and °C.

3. Parts and Controls


Included in Package:

- Infrared ear thermometer
- Storage box
- English/Spanish instruction manual
- English/Spanish Quick start guide
- 2 x 1.5V AAA batteries
- 10 Protective caps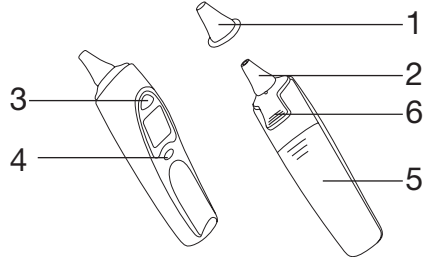

- 1 Protective Cover
- 2 Sensor
- 3 START Button
- 4 ON/OFF Button "⏻"
- 5 Battery Compartment
- 6 Protective Cover Removal Lever

- 7 Ambient Temperature Mode Indicator
- 8 Room Temperature/Time/Date
- 9 AM/PM Indicators
- 10 Battery Level Indicator
- 11 Memory Mode
- 12 Normal Temperature Indicator
- 13 Temperature Scale Indicator (°C or °F)
- 14 Fever Indicator
- 15 Temperature Display
- 16 Surface/Object Temperature Mode Indicator
- 17 Protective Cover Indicator
- 18 Body Temperature Mode Indicator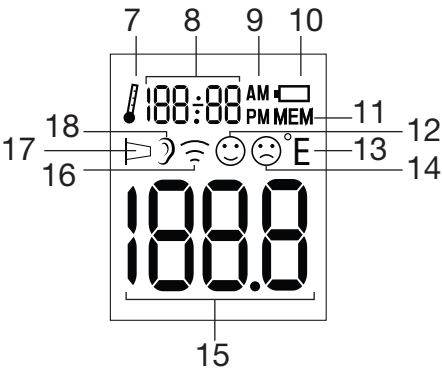


4. To Set Up



Before first use, remove the clear battery insulating strip from the Battery Compartment by pulling it out gently; the device will power itself on.

Setting the time and date:

1. With the device switched off, press and hold the "⏻" Button for three seconds; "12 H" will appear on the LCD Display.	12 H
2. Use the START Button to select 12 or 24-hour time display format; press the "⏻" Button to save and continue.	12 H 24 H

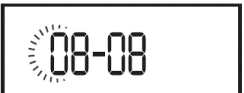3. Continue in this fashion to set hour, minute, year, month, and day in turn.	
4. After the day is set, press the "⏻" Button to save and exit; the device will shut off.	

Selecting a temperature scale (°F or °C):

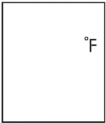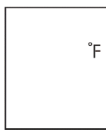1. With the device switched off, press and hold the START button for three seconds; "°F" appears in the LCD Display.	
2. Press the START Button to select °F or °C and then press the "⏻" Button to save and exit.	 

Changing batteries:

Two AAA batteries are pre-installed and should last for about 3,000 measurements. When battery power gets low, the low battery icon "🔋" appears. Replace the batteries as soon as the battery symbol "🔋" appears on the display.


1. Shut the thermometer off.
2. Slide the Battery Compartment Cover toward the bottom of the thermometer to remove.
3. Remove the old batteries and replace with two new AAA alkaline batteries, following the polarity markings inside the compartment. Reattach the Battery Compartment Cover.

5. To Use


NOTE:

- Check before each use that the lens is intact. If it is damaged, contact your retailer or the service address. Bear in mind that the ear thermometer needs to have been in the room in which the measurement is taken for at least 30 minutes before use.


Ear Temperature Measurement:

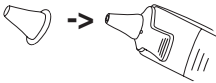1. Place a clean Protective Cover onto the sensor.	
2. Press the ON/OFF Button "⏻" to switch the thermometer on; it will beep three times and the LCD Display will activate.	


3. The ear canal is slightly curved. Pull the ear slightly up and backwards and then insert the sensor so the tip of the protective cover points directly at the eardrum.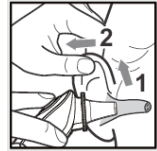

4. With the sensor properly in place, press the START Button.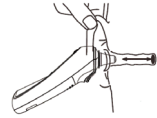

5. After about a second, the thermometer will beep once and display the measurement. Press the ON/OFF Button to shut the thermometer off.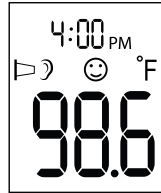

NOTE: If no buttons are pressed for one minute, the thermometer will shut off.

Fever indicator:


If the measured temperature is below 100.4 °F (38 °C), a smiley face “☺” appears in the LCD Display along with a short beep. If the measured temperature is 99.5–100.3 °F (37.5–37.9 °C), two short beeps will also sound, indicating a slightly raised temperature. If the measured temperature is 100.4 °F (38 °C) or greater, a sad face “☹” appears in the LCD Display and the device will emit one short, one long, and three short beeps.

Surface temperatures:


To measure surface temperatures, change the thermometer to Surface Temperature Mode.

1. Remove the Protective Cover from the sensor.

2. Press the ON/OFF Button to turn the thermometer on.

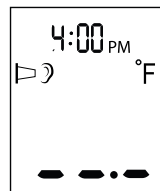
3. Change the thermometer to surface temperature mode by pressing and holding the ON/OFF Button and START Button at the same time. The surface temperature mode icon “☺” will display.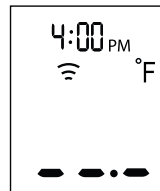


4. Hold the sensor tip of the ear thermometer 0.4-0.8 inch (1–2 cm) from the surface to be measured and press the START Button.

5. After one second, the thermometer will beep once and display the measurement. Press the ON/OFF button to shut the thermometer off.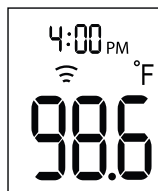

Ambient temperature:


When the thermometer powers on, the ambient temperature, date, and time will alternate on the LCD Display.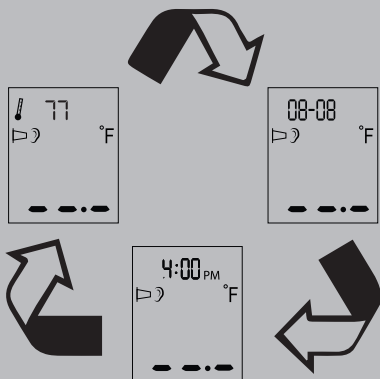

Displaying stored measurements:

The ear thermometer stores the last 10 ear and surface temperatures, along with the measurement date and time. To recall stored measurements, proceed as follows:


1. Press the ON/OFF button to switch the thermometer on.

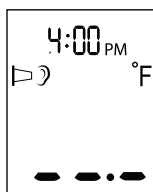2. Press and hold the ON/OFF button for three seconds; "MEM" and the last stored measurement will appear on the LCD Display.


3. To view stored measurements, press the ON/OFF Button to cycle; the memory number is displayed first followed by the measurement.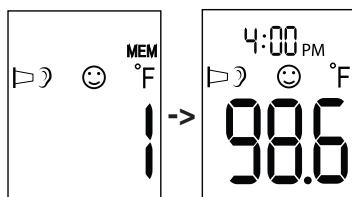

4. To proceed with a new measurement, press the START Button; otherwise, press and hold the ON/OFF button for three seconds to shut the thermometer off.

6. Care and Maintenance

- Protective Covers are intended for a single use only. Discard after each use.
- The sensor is the most sensitive part of the thermometer. Work carefully when cleaning the sensor to avoid electrical damage. Use a clean cloth or cotton swab moistened with disinfectant, alcohol, or warm water.
- To clean the outer surface of the thermometer, use a soft cloth slightly moistened with a mild soapy solution.
- Do not use any harsh cleaning products.
- Before prolonged storage, remove the batteries.
- Do not use or store the device in excessively high or low temperature or humidity (see technical data), in direct sunlight, near an electrical current or in dusty locations.
- Clean the sensor tip after every use. Use a clean cloth or cotton swab that can be moistened with rubbing alcohol or warm water.
- To clean the device, use a soft, damp cloth and a mild detergent. Ensure water does not get into the device. If the device does get wet, remove the battery immediately and let dry completely before reuse.
- Never immerse the device in water.
- To protect the sensor tip, always replace protective cap after use.
- When storing the device for an extended period, remove the battery.

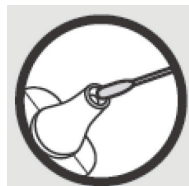7. Troubleshooting Guide

Error message	Problem	Solution
Er 1	Autotest in progress, device not ready.	Retry measurement. Take measurements only once body (👤) or object temperature mode (🌡️) appears on the display.
Er 2	Ambient temperature is not within measurement range.	Place the thermometer in an ambient temperature between 50° F and 104° F (10° C - 40° C) for at least 30 minutes.
Er 3	Thermometer not placed in ear correctly.	Please observe the notes in the chapter "Measuring". Perform the measurement again.
Er 4	Large fluctuation in ambient temperature.	Store the device for at least 30 minutes in the room where the measurement will be taken. Place the ear thermometer in a room with an ambient temperature between 10 °C and 40 °C (50 °F and 104 °F) for at least 30 minutes.

E-5	Thermometer is not working properly.	Replace batteries and retry. If problems persist, contact customer service.
E-6	Thermometer is in Surface Temperature mode and Protective Cover is covering sensor.	Press the Protective Cover Removal Lever to remove cover.
H_i	In ear measurement: the measured temperature is over 109.4 °F (43 °C). Object measurement: the measured temperature is over 212 °F (100 °C).	Only use thermometer within measurement range. Ensure that the device and the protective cap are fully functioning. Perform the measurement again.
L_o	In ear measurement: the measured temperature is below 82.4 °F (28 °C). Object measurement: the measured temperature is below -4 °F (-20 °C).	Only use thermometer within measurement range. Ensure that the device and the protective cap are fully functioning. Perform the measurement again.
	Batteries low or depleted.	Replace batteries.

8. Specifications

Temperature Measurement Range	Ear: 82.4 °F - 109.4 °F (28.0 °C - 43.0 °C) Surface temperature: -4 °F - 212 °F (-20 °C - 100 °C) Ambient temperature: 32 °F - 122 °F (0 °C - 50 °C)
Measurement Accuracy	Ear: ±0.4 °F (0.2 °C) from 95.9 °F - 107.6 °F (35.5 °C - 42.0 °C); ±0.5 °F (0.3 °C) below 95.9 °F (35.5 °C) or above 107.6 °F (42.0 °C) Object temperature: ±4 °F (2 °C) or 4% Room temperature: ±4 °F (2 °C)
Measurement Duration	Approx. one second
Operating conditions	50 °F- 104 °F (10 °C-40 °C), Humidity 15 - 85% (noncondensing)
Storage conditions	-13 °F - 131 °F (-25 °C - 55 °C), Humidity 15 - 95% (noncondensing)
Memory Slots	10
Display resolution	0.1 °F or 0.1 °C
Power	Two AAA batteries
Dimensions	6.1" x 1.5" x 1.9" (155 x 38 x 47 mm)
Weight	Approx. 2.2 oz (63 g) without batteries
Automatic shutoff	60 seconds
Battery life time	3000 Measurements or one year with one to two measurements per day
	Application part, type BF
IP 22	IP22, protection against harmful ingress of water and dust

9. FCC Compliance Declaration

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

10. Warranty

Limited Lifetime Warranty For Original Purchaser

Your beurer ear thermometer, Model FT58, is warranted to be free from defects in materials and workmanship for the life of the product under normal conditions of intended use and service. This warranty extends only to the original retail purchaser and does not extend to retailers or subsequent owners.

We will, at our option, repair or replace the beurer ear thermometer, Model FT58, without additional charge, for any part or parts covered by these written warranties. No refunds will be given. Repair or replacement is our only responsibility and your only remedy under this written warranty. If replacement parts for defective materials are not available, Beurer reserves the right to make product substitutions in lieu of repair or replacement. For warranty service contact our customer service department at **1-800-536-0366** or at **info@beurer.com** to provide a description of the problem. If the problem is deemed to be within the scope of the limited lifetime warranty, you will be asked to mail the product at your costs in its original package with proof of purchase, your name, address and phone number. If the problem is not deemed to be within the scope of the limited lifetime warranty, we will provide a quotation for repair respectively replacement and return shipping fee.

This warranty does not cover damage caused by misuse or abuse; accident; the attachment of unauthorized accessory; alteration to the product; improper installation; misapplication; lack of reasonable care with respect to the product; unauthorized repairs or modifications; improper use of electrical/power supply; normal wear; loss of power; dropped product; malfunction or damage of an operating part as a result of failure to comply with instructions for use or to provide manufacturer's recommended maintenance; transit damage; theft; neglect; vandalism; or environmental conditions; loss of use during the period the product is at a repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of Beurer. This warranty is void if the product is ever used in a commercial or business environment. The maximum liability of Beurer under this warranty is limited to the purchase price actually paid by the customer for the product covered by the warranty, as confirmed by proof of purchase, regardless of the amount of any other direct or indirect damage suffered by the customer.

This warranty is effective only if the product is purchased and operated in the country in which the product is purchased. A product that requires modifications or adaptation to enable it to operate in any other country than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under this warranty.

THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY. ANY IMPLIED WARRANTIES, OBLIGATIONS, OR LIABILITIES, INCLUDING BUT NOT LIMITED TO THE IMPLIED WARRANTY OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE LIMITED IN DURATION TO THE DURATION OF THIS APPLICABLE WRITTEN WARRANTY. Some states do not allow limitations on how long an implied warranty lasts, so the above limitations may not apply to you.

IN NO EVENT SHALL BEURER BE LIABLE FOR ANY SPECIAL, INCIDENTAL, INDIRECT OR CONSEQUENTIAL DAMAGES FOR BREACH OF THIS OR ANY OTHER WARRANTY, EXPRESS, IMPLIED OR

ANY OTHER THEORY OF LIABILITY, WHATSOEVER. Some states do not allow the exclusion or limitation of special, incidental, or consequential damages, so the above limitation may not apply to you. Beurer does not authorize anyone, including, but not limited to, retailers, the subsequent consumer purchaser of the product from a retailer or remote purchaser, to obligate Beurer in any way beyond the terms set forth herein.

This warranty does not extend to the purchase of opened, used, repaired, repackaged and/or resealed products, including but not limited to sale of such products on Internet auction sites and/or products by surplus or bulk resellers. Any and all warranties or guarantees shall immediately cease and terminate in connection with any products or parts thereof which are repaired, replaced, altered, or modified, without the prior explicitly written consent of Beurer.

This warranty gives you specific legal rights, and you may also have other rights which may vary from state to state.

Questions or comments? Call toll free 1-800-536-0366 or contact info@beurer.com

Manufactured for:
Beurer North America LP,
900 N Federal Hwy Ste 300
Hallandale Beach, FL 33009 USA
Made in China

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